ultraMEDIC[®]

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Contents

Page

1. Product description	4
2. Important information	12
2.1 Liability and warranty	12
2.2 User group	12
2.3 Declaration of conformity	12
3. Information on the safety and protection of the product	12
4. Intended use	13
4.1 Cautions	13
4.2 Application as scoop stretcher	13
4.3 Application as spine board	16
5. Cleaning and care	16
6. Maintenance	16
7. Storage	17
8. Service life	17
9. Optional accessories	17
10. Identification and warranty certificate	18

2. Important information

Please read the instructions for use and safety instructions carefully before you start using the COMBI-STRETCHER II. Keep the instructions for use in a safe place to avoid operating errors.

Should you require further information or encounter difficulties in its use or maintenance, please contact your local dealer or ultraMEDIC GmbH directly. Always quote the LOT- or serial number to identify the COMBI-STRETCHER II.

2.1 Liability and warranty

The manufacturer grants a warranty period of 24 months on this product. The warranty only covers the costs of the spare parts. Any incurred shipping or wage costs are not subject to warranty.

Please check the shipment for damage and completeness immediately upon opening. If damage is found, this must be reported within two working days, packaging damage must be reported immediately. After this period, no guarantee or warranty claims for transport damage can be asserted. No responsibility is taken for improper assembly.

2.2 User group

This product may only be used by competent, trained and qualified personnel, and the principles of use must be observed. A prerequisite for use is theoretical and practical experience in handling the COMBI-STRETCHER II.

2.3 Declaration of conformity

The ultraCOMBI-STRETCHER II was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

3. Information on the safety and protection of the product

1. Before and after every use, check that the components and safety belts are intact and complete.
2. Make sure that the safety belts and components are assembled correctly.
3. Before using the COMBI-STRETCHER II for the first time, conduct a test

run to familiarize yourself with the product.

4. In the event of damage or suspected damage to the safety belts or COMBI-STRETCHER II remove the product from operation immediately until any occurring irregularities have been remedied.

4. Intended use

The ultraCOMBI-STRETCHER II is a combined spine board and scoop stretcher used to immobilise injured persons who may have suffered a spinal fracture.

4.1 Cautions

1. A specialist medical assessment must be conducted before the injured party is moved, lifted or transported.
2. To ensure the patient's safety, at least 2 users are required to operate the COMBI-STRETCHER II.
3. The injured party must be immobilised on the COMBI-STRETCHER II during every use. Insufficient or no restraint may lead to severe patient or rescuer injuries.
4. Serious incidents that have occurred in connection with the product must be reported by the user and / or patient to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

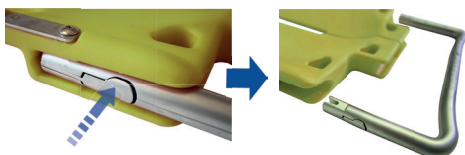
Fig. 1
metal bracket and pull

4.2 Application as scoop stretcher

1. To position the ultraCOMBI-STRETCHER II under the patient without moving it, open one of the metal brackets. To do this, press the metal button on the side of the out the metal bracket (Fig. 1).

There are various application techniques for the scoop stretcher. You can place the halves separately laterally under the patient, then carefully position the headboard under the patient's head and close the footboard

(so-called "scooping" motion).

The type of application must be individually determined, depending on the nature of the injury.

2. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.
3. When positioning Longitudinal Parts under the patient, make sure that no body parts are trapped between the Longitudinal Parts.
4. Reconnect the two parts of the ultraCOMBI-STRETCHER II to the metal brackets and make sure that the metal brackets snap into place. After closing the parts (clearly audible clicking sound when closing), make sure that both snap locks of the ultraCOMBI-STRETCHER II are closed, only then fix the patient with the patient safety belts.
5. Secure all 3 patient safety belts (Figure 2). The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the belts.
6. Check all buckles and belts before lifting and transporting the patient.

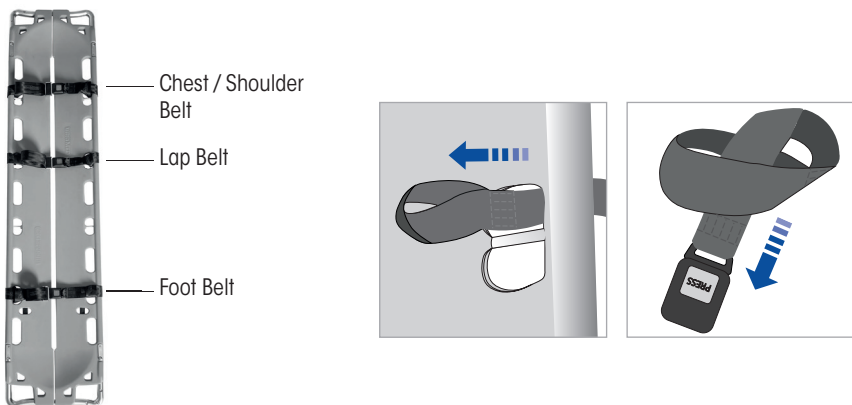
Fig. 2

Fastening of patient safety belts

Application example with 3 first responders:

- The 1 first aider is on his/het knees on the side of the injured person's head and secures the injured person. This first aider guides and directs the entire manoeuvre.
- The other two first responders are to the right and left of the injured person.

Prior to the lifting manoeuvre:

- Place the ultraCOMBI-STRETCHER II next to the injured person. As a reference point, take the top of the injured person's head to ensure the head-neck-trunk axis.
- Remove the metal brackets and open the ultraCOMBI-STRETCHER II completely. The metal brackets can remain attached to one of the Longitudinal Parts depending on the application situation.
- If a spinal injury is suspected, stabilize the cervical spine of the injured person. (The ultraCOMBI-STRETCHER II can be used with the optional accessory ultraHEAD-LOCK III. The head fixation set ultraHEAD-LOCK III can be ordered separately under item number SAN-0291.)
- Place the palms of the injured person on their thighs to avoid pinching the arms when closing the ultraCOMBI-STRETCHER II.

Further procedure for 2nd and 3rd first responders:

- (First aiders 2 and 3 stay on either side of the injured person, each with a longitudinal part of the ultraCOMBI-STRETCHER II.) Take turns sliding a Longitudinal Part under the casualty. While the Longitudinal Part is being slid in, the other person grasps the casualty at the shoulder and hip and pulls him or her towards them, enough to relieve the pressure and make it easier to place the blade under the casualty. When placing the second blade, make sure it is facing the other one to facilitate its closure.
- Join and secure the two parts of the ultraCOMBI-STRETCHER II using the metal brackets and ensure that they click into place correctly. **Start with the one at the head, then the one at the feet.**
- If the casualty is tall, it may be necessary to lift the feet slightly to ensure the device closes.
- Ensure that the snap-on device is closed by pulling the two parts of the ultraCOMBI-STRETCHER II sideways.
- Secure the casualty with all 3 patient safety belts on the ultraCOMBI-STRETCHER II. The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the belts.

4.3 Application as spine board

1. Before using the ultraCOMBI-STRETCHER II as a spine board, ensure that both snap buckles are locked.
2. Similar to a spine board, the ultraCOMBI-STRETCHER is applied depending on the injury and local conditions.
3. If the injured person is shorter than 1.20 m, we recommend using the straps SAN-0281 or SAN-0286.
4. Secure the casualty with all 3 patient safety belts on the ultraCOMBI-STRETCHER II. The latches must always audibly lock into place. For safety reasons, subject the latches to a test by pulling at the appropriate point. Ensure that the injured party's breathing is not encumbered by the traction of the belts.
5. Check all buckles and belts before lifting and transporting the patient.

5. Cleaning and care

Use mild soap and warm water (40°C) to clean the COMBI-STRETCHER II.

For disinfection, you can use commercially available, alcohol-based agents. Other solutions or cleaning chemicals may damage the material ultraCOMBI-STRETCHERS II and of the belts.

The snap locks must not be removed for cleaning. After cleaning with a mild soap and warm water, the closures do not require additional lubrication.

6. Maintenance

In order to ensure the safety of the injured party and the user, we recommend that you have the COMBI-STRETCHER II visually inspected by an expert at least once a year. Appointed auditors must be demonstrably adequately qualified.

For users in Germany, we recommend consideration of the following regulations and guidelines:

DGUV Grundsatz 305-002
DGUV Regel 112-198/199
MPBetreibV

The following parts must be checked and the following defects must be excluded:

Base plate: deformation, tears, breaks, loosened screws, broken parts

Closures: impaired function, deformation, torsion, tears and breaks

Belts: Cutting and abrasion points, damaged seams, tissue-damaging stains

Metal parts: Rust and oxidation

other damages

In the event of damage or suspected damage, remove the product from operation immediately.

7. Storage

Store the COMBI-STRETCHER II in a dry and dark place. Avoid direct sunlight and direct heat radiation from heat sources. These factors could damage the material.

Do not place heavy objects on the device during storage.

We recommend storing the ultraCOMBI-STRETCHER II as one unit to avoid loss of individual parts and possible damage.

8. Service life

The discard criteria depends on the product, its frequency of use and the external conditions of use. The service life of the COMBI-STRETCHER II depends on the individual conditions of use. Depending on the frequency and intensity of use of the transportation suspension, certain wear and tear is normal. Appropriate use is assumed here.

Service life of the **patient safety belts (SAN-0084-S / SAN-0084-K)** and of the **safety strap systems (SAN-0286 / SAN-0281)** is set at max. **10 years**.

The service life of the optional accessories can be found in the associated operating instructions.

9. Optional accessories

SAN-0291	ultraHEAD-LOCK III	Head fixation set
SAN-0286	ultraSTRAPS Xtreme	Safety strap system with clip locks
SAN-0281	ultraSTRAPS	Safety strap system with Velcro fastener
SAN-0280-T	ultraSPINE BAG	Storage bag
SAN-0084-S	ultraBELT	Replacement safety belt
SAN-0084-K	ultraBELT SET	3 Replacement safety belts incl. storage bag
SAN-0260-2	X-BOON TWO	Vacuum mattress
SAN-0278	ultraVACPUMP I	Vacuum pump

10. Identifizierungs- und Gewährleistungszertifikat
Identification and Warranty Certificate
Certificat d'identification et de garantie
Certificado de identificación y de garantia

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