216 | Model D Loop | Gebrauchsanweisung / Instructions for Use / Notice d'utilisation

Deutsch

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Product name: Model D Loop

Product reference: 216 216-5

Category: Medical device class I according to Regulation (EU) 2017/745. It meets the requirements of EN 14683:2019+AC:2019 for medical face masks as well as the biological evaluation of medical devices according to EN ISO 10993.

Intended purpose: Medical face masks are manufactured and indicated for third-party protection and protect the counterpart from exposure of potentially infectious droplets of the person wearing the medical face mask.

Indication: Use in nosocomial and non-nosocomial areas for prophylaxis against infectious agents.

Contraindication: Masks may be unsuitable for users who have received an appropriate indication from their doctor e.g. due to an existing restriction or disease of the respiratory tract. The mask contains metal parts in the nosepiece, therefore it is not suitable for use in strong magnetic fields or potentially explosive atmospheres.

Intended user: Medical and nursing staff and end users especially during a pandemic.

Use:

- Suitable for single use.
- Used masks are considered contaminated
- Do not clean or disinfect
- Product is non-sterile

Incident notification: All serious incidents related to the product must be immediately reported to the manufacturer and the competent authority.

Storage: Storage in dry environment and protected from direct sunlight in the original packaging see pictograms on the packaging.

Storage period: If the storage conditions are observed the product has a storage period of 5 years (see labelling on the product).

Disposal: Dispose the mask immediately in a proper way i.e. in a closed container.

CHECK BEFORE USE: Inspect the packaging and the mask including all components for holes tears or damage before use. If any damage is found, DO NOT use the mask.

216 | Model D Loop | Gebrauchsanweisung / Instructions for Use / Notice d'utilisation

Dressing instructions:

- Before touching the mask, wash your hands with soap and water or clean your hands with an alcohol-based hand sanitizer.
- Hold the mask in front of your face: the coloured side facing out and the edge with the nose wire facing up.
- Pick up the ear loops and pull them behind your ears. Bend the nose loop around the nose with both hands to get a good fit on the face.
- As you are holding the mask in place with one hand, pull the underside of the mask under your chin to pull the folds of the mask apart. The mask should now completely cover your nose mouth and jaw.
- If you wear glasses, place them over the mask.

IMPORTANT: Do not touch the contaminated outside of the mask during or after use. Pull the ear loops off the ears to remove the mask. Wash and disinfect your hands after use. If the mask is wet replace it immediately

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